

**REAADS Anti-Phosphatidylserine  
Test Kits**  
*For In Vitro Diagnostic Use*

**REAADS  
Anti-Phosphatidylserine IgG/IgM  
Test Kit**

Product #: 030-001  
(96 well kit)  
Product #: 030-002  
(288 well kit)

**REAADS  
Anti-Phosphatidylserine IgA  
Test Kit**

Product #: 10206  
(96 well kit)  
Product #: 10497  
(288 well kit)

- **Convenient ELISA Procedure**
- **For the specific determination of IgG, IgM or IgA isotypes**
- **Objective, accurate and reproducible**
- **Reagent complete kits**
- **Total incubation time: 40 minutes at room temperature**
- **Easily Automated**
- **Choice of single or multi-point calibration**

**Background**

Elevated serum levels of anti-phospholipid antibodies are associated with an increased risk for recurrent arterial and venous thrombotic events, thrombocytopenia and fetal loss. These are the main clinical manifestations of Antiphospholipid

Syndrome. Phosphatidylserine is found in the membranes of platelets and endothelial cells, which participate in the coagulation cascade. Due to its physiologic role, testing for autoantibodies directed towards phosphatidylserine provides not only more relevant results, but also additional information to assess the risk of thrombosis.

**Principle**

The REAADS IgG, IgM and IgA Anti-Phosphatidylserine (aPS) Test Kits are indirect, semi-quantitative ELISAs used to detect antibodies against phosphatidylserine. Each assay uses a single or multi point calibrator to obtain IgG, IgM or IgA anti-phosphatidylserine antibody concentrations from patient samples.

**Procedure**

Patient serum is diluted into sample diluent containing bovine serum (source of cofactor) and incubated in wells coated with phosphatidylserine.

Any aPS antibodies present in the patient sample will bind to the coated wells. After removal of unbound serum by washing, antibodies specific for human IgG, IgM or IgA labeled with horseradish peroxidase (HRP), are added to form complexes with the phosphatidylserine bound antibodies. The wells are washed, and a chromogenic substrate is added, resulting in a color change that is proportional to the amount of antibody present. The assay is stopped by the addition of a stopping solution. Results are obtained by reading the O.D. (optical density) of each well at 450nm in a spectrophotometer. The O.D. values of control and patient samples are multiplied by the calibrator conversion factor to obtain IgG, IgM or IgA anti-phosphatidylserine antibody concentrations in GPS, MPS or APS units, respectively. Assay cutoffs have been established at 16 GPS, 22 MPS and 20 APS units.

**READS Anti-Phosphatidylserine  
Test Kits  
For In Vitro Diagnostic Use**

## Clinical Performance

Clinical Specificity: serum samples from multiple healthy blood donor populations were assayed for the presence of anti-phosphatidylserine antibodies. Specificity was shown to be 96% for both IgG and IgM antibodies, and 95% for IgA.

Clinical Sensitivity: unselected SLE samples were tested to determine the clinical sensitivity of the aPS assay. The resulting sensitivity was 22.2% for IgG, 25% for IgM and 11.1% for IgA. The clinical sensitivity of the assay for Antiphospholipid Syndrome (APS) was determined by comparing aPS test results from two groups of selected patients: Group 1 - patients with Primary APS; and Group 2 - SLE patients with no history of thrombosis or thrombocytopenia (controls). The results are shown in the chart above.

The prevalence and mean levels of aPS antibodies were higher in Primary APS patients than in the SLE control group.

These assays, when run in concert with additional anti-phospholipid and lupus anticoagulant assays, provides a more complete profile for establishing a diagnosis of Antiphospholipid Syndrome and for assessing the risk of thrombotic complications in SLE patients.

### Group 1 - Primary APS

	IgG aPS	IgM aPS	IgA aPS
<b>Assay cutoff</b>	<b>16 GPS</b>	<b>22 MPS</b>	<b>20 APS</b>
<b>Average value</b>	<b>61.6 GPS</b>	<b>33.4 MPS</b>	<b>21.1 APS</b>
<b>% positive</b>	<b>90.9%</b>	<b>54.5%</b>	<b>36.4%</b>

### Group 2 - SLE control

	IgG aPS	IgM aPS	IgA aPS
<b>Assay cutoff</b>	<b>16 GPS</b>	<b>22 MPS</b>	<b>20 APS</b>
<b>Average value</b>	<b>9.2 GPS</b>	<b>9.9 MPS</b>	<b>11.2 APS</b>
<b>% positive</b>	<b>11.1%</b>	<b>11.1%</b>	<b>0%</b>



#### **Corgenix, Inc.**

12061 Tejon Street  
Westminster, Colorado 80234  
Toll Free: 800-729-5661  
Telephone: 303-457-4345  
Fax: 303-457-4519  
Email: techsupport@corgenix.com  
www.corgenixonline.com

#### **Corgenix (UK) Ltd.**

75 Broadway  
Peterborough, PE1 1SY UK  
Telephone: +44 (0) 1733 296800  
Fax: +44 (0) 1733 296809  
Email: sales@corgenix.co.uk  
www.corgenix.co.uk

Corgenix Medical Corporation trades on the NASD (OTC BB) under the ticker symbol CONX. Corgenix, Inc. operates under FDA Quality System Regulations and is ISO 9001:1994/ISO 13485/EN 46001 certified.